ES

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NO CONGELAR. Proteger da luz solar directa.

Preservação and stability

The kit must be stored at a temperature between 2 and 30°C. Under these conditions, we can guarantee the stability until the expiry date printed on the box and on the individual pouch.

The kit has been developed to be stored at room temperature. Although it can also be stored in the refrigerator, we recommend store it at room temperature to avoid the need to wait for reagents to reach the room temperature.

DO NOT FREEZE. Do not expose to direct sunlight.

Conservation and stability

The kit must be stored at a temperature between 2 and 30°C. Under these conditions, we can guarantee the stability until the expiry date printed on the box and on the individual pouch.

The kit has been developed to be stored at room temperature. Although it can also be stored in the refrigerator, we recommend store it at room temperature to avoid the need to wait for reagents to reach the room temperature.

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For veterinary use only

Technical basis

The UranoVet® Parvo-Corona diagnostic kit is based on the immunochemical technique and is designed for the qualitative detection of Canine parvovirus and canine coronavirus in faeces.

The test has a double structure. It contains two single tests: a strip for CPV antigen detection and a strip for CCV antigen detection. Each test strip consists of a test area (T line) and a control area (C line). The lines are not visible before applying the sample. After applying the sample in the appropriate sample well, migration begins by capillary action through the membrane. If the result is negative, one purple colour band appears in the C area. This line, called control line, always appears as a control line indicating that the test has successfully performed if the result test is positive, in addition to the control line, a second line will form in the test area (T line).

Materials supplied

- 1 - Double test devices individually packaged in aluminium pouch.
- 2 - Tubes with buffer solution for sample dilution.
- 3 - Swabs for sample collection.
- 4 - Disposable pipettes.
- 5 - Instructions for use.

Precautions

- 1 - For veterinary use only.
- 2 - Wear disposable gloves when handling the samples. All samples should be treated as potentially infectious. Wash and disinfect hands after dispensing the sample.
- 3 - Do not reuse the test device just before use.
- 4 - All reagents must be at room temperature before performing the test.
- 5 - Do not use the test if the envelope is damaged or broken.
- 6 - Do not reuse reagents after the expiry date.
- 7 - Do not use reagents after the expiry date.
- 8 - Do not mix components or reagents from kits with different batch numbers.

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Preservation and stability

1 - For veterinary use only.

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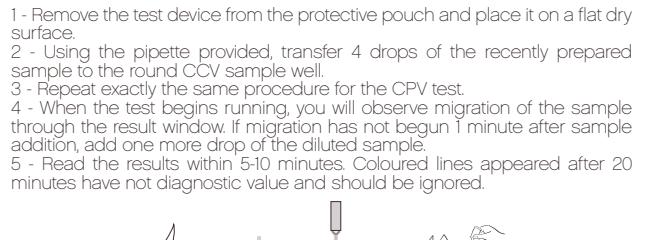
Sample collection and preparation

The sample may be collected directly from the rectum using the provided swab. The sufficient amount of sample to carry out properly the test is shown in the picture.

Introduce the swab into the tube containing the diluent and press it against the tube walls in order to release the sample that could contain the virus. Shake the tube in order to ensure a good homogenisation.

**Instructions for use**

- 1 - Remove the test device from the protective pouch and place it on a flat dry surface.
- 2 - Using the pipette provided, transfer 4 drops of the recently prepared swab. The sufficient amount of sample to carry out properly the test is shown in the picture.
- 3 - Repeat exactly the same procedure for the CPV test.
- 4 - When the test begins running, you will observe migration of the sample through the result window. If migration has not begun 1 minute after sample addition, add one more drop of the diluted sample.
- 5 - Read the results within 5-10 minutes. Coloured lines appeared after 20 minutes have not diagnostic value and should be ignored.

**Interpreting results**

- 1 - Negative result
There is one single line (control line) in the C-zone in the result window, both in the Parvovirus determination zone and in the Coronavirus determination zone.

- 2 - Positive result
Two lines appear in either or both of the windows.

- 3 - Invalid result
The test is invalid if no coloured line appears in the C area even if a coloured line appears in the T area. The reason may be due to incorrect handling or using a damaged test.

**Limitations of the technique**

Even though the UranoVet® Parvo-Corona test kit shows high sensitivity and specificity, cannot be excluded a low incidence of false positive or negative results. As any other laboratory procedure, the definitive clinical diagnosis cannot be based only on the test result. It must be based on an ensemble of clinical and laboratory procedures. If there is any doubt, repeat the test and/or contrast with other diagnostic methods.

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